

Multicenter Evaluation of Cephalexin MIC Results for *Enterobacteriales* Using EUCAST Breakpoints on MicroScan Dried Gram Negative MIC Panels

O.B. Garner¹, C. Emery², A. Harrington³, S. DesJarlais³, C. J. Haste⁴, R.K. Brookman⁴, Z.C. Lockett⁴, and J.Y. Chau⁴

¹UCLA David Geffen School of Medicine, Los Angeles, CA, ²Indiana University School of Medicine, Indianapolis, IN, ³Loyola University Medical Center, Maywood, IL, and ⁴Beckman Coulter Microbiology, West Sacramento, CA

Background: A multicenter study was performed to evaluate the accuracy of cephalexin on a MicroScan Dried Gram-negative MIC (MSDGN) Panel when compared to a frozen ISO/CLSI broth microdilution reference panel.

Materials/Methods: An evaluation was conducted at three U.S. sites by comparing MIC values obtained using the MSDGN to MICs utilizing an ISO/CLSI broth microdilution reference panel. A total of 450 *Enterobacteriales* clinical isolates were tested using the turbidity and Prompt® methods of inoculation during the efficacy phase. A subset of 12 organisms was tested on MSDGN panels at each site during reproducibility. MSDGN panels were incubated at 35 ± 1°C and read on the WalkAway System, the autoSCAN-4 instrument, and read visually. Read times for the MSDGN panels were at 16-20 hours. Frozen reference panels were prepared according to ISO/CLSI methodology, incubated for 16-20 hours and read visually. All frozen reference panels were incubated at 35 ± 2°C for 16-20 hours and read visually. EUCAST breakpoints (mg/L) used for interpretation of MIC results were: *Enterobacteriales* ≤ 16 S, > 16 R.

Results: Essential and categorical agreement when compared to frozen reference panel results, for all isolates tested in efficacy as follows:

Read Method	Essential Agreement %		Categorical Agreement %		Very Major Error (VMJ)* %		Major Error (MAJ)* %	
	P	T	P	T	P	T	P	T
WalkAway	98.0 (441/450)	98.4 (443/450)	96.7 (435/450)	97.3 (438/450)	0.0 (0/188)	0.0 (0/188)	1.5 (4/262)	0.8 (2/262)
autoSCAN-4	97.8 (440/450)	98.7 (444/450)	96.9 (436/450)	98.2 (442/450)	0.0 (0/188)	0.0 (0/188)	1.5 (4/262)	0.0 (0/262)
Visually	98.0 (441/450)	98.7 (444/450)	97.1 (437/450)	98.0 (441/450)	0.0 (0/188)	0.0 (0/188)	1.5 (4/262)	0.4 (1/262)
T = Turbidity inoculation method, P = Prompt inoculation method								
*Calculation of MAJ and VMJ excluding 1 well errors								

Conclusion: This multicenter study showed that cephalexin MIC results for *Enterobacteriales* obtained with the MSDGN panel correlate well with MICs obtained using frozen reference panels using EUCAST interpretive criteria.

PROMPT® is a registered trademark of 3M Company, St. Paul, MN USA.

Beckman Coulter, the stylized logo and the Beckman Coulter product and service marks mentioned herein are trademarks or registered trademarks of Beckman Coulter, Inc. in the United States and other countries.