



INDUSTRY-PROVEN RELIABILITY AND PRODUCTIVITY

PK7400 Automated Microplate System

Unmatched throughput. Simple to use. Reliable results.




MARKET LEADER IN ANALYZER THROUGHPUT WITH A HISTORY OF RELIABILITY FOR BLOOD-DONOR IMMUNOHEMATOLOGY TESTING

Results you can rely on

Experience the reliability and efficiency of the industry-leading PK7400 Automated Microplate System—from your global partner in high-volume donor-center testing solutions. For over 30 years, Beckman Coulter's PK systems have been synonymous with dependability. Today's high-quality PK7400 system is no exception, offering the accuracy and workload capacity customers have come to know and respect.

- > Experience how the terraced microplate—designed specifically for hemagglutination—increases the reliability of each test within the following menu:
 - ABO/Rh
 - Red blood cell screening
 - Cytomegalovirus (CMV)
 - Syphilis (TP)
- > Boost testing efficiency with the highest throughput available of any high-volume donor-center system
- > Benefit from the convenience and customer-focused care provided by knowledgeable service and support representatives

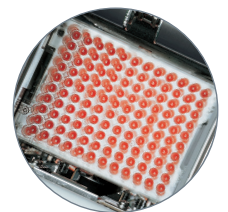

PK7400 Automated Microplate System

Simplicity and efficiency

High throughput:

Up to 300 samples per hour allows rapid analysis of large sample volumes as well as the ability to test multiple assays per sample.

User-friendly software:

Customized menus, icons and color-keyed graphics reduce training time.

Graphical user interface:

The new operating software provides improved access and visibility to the most common functional areas: reagent management, online maintenance and daily startup tasks.

A new submenu space can be used to display error messages, customized user menus, event details or serve as a blank notepad for operator notes.

And the new graphical user interface achieves a consistent and recognizable user experience across Beckman Coulter instrumentation product lines.

Programmable start-up:

Programmable, fully automated startup mode saves time and promotes workflow efficiency.

Onboard data management:

Store and search test results. Data can also be saved for off-line analysis and archiving or transmitted to a host LIS.

Quality and reliability

Analysis process monitoring:

Dispensed number of samples, reagents and diluents are continuously monitored. Anomalies are automatically detected, displayed and reported.

Stable reaction environment:

A constant reaction environment is maintained and monitored within the incubator to continuously assess system processes.

Reliable analysis using a high-resolution color CCD camera:

The reaction image and assessment results are displayed in color on the monitor, then stored as image data.

Enhanced test reliability using ID management:

Barcodes provide automated management of samples, microplates, reagents and diluents.



UNMATCHED THROUGHPUT



SIMPLE TO USE



RELIABLE RESULTS



PK7400 AUTOMATED MICROPLATE SYSTEM

Specifications

Electrical Requirements:

Electrical consumption:	3.0 KVA maximum
Current:	15~30 amp (with UPS)
Circuit:	Dedicated and noise-free
Voltage:	200/208/220/230/240 VAC (±10%) Single phase
Frequency:	50/60 Hz (±1 Hz)
Ground requirement:	<100 ohms
Location:	Power cable length 10 m (30 ft)

Water Requirements:

Type:	Deionized
Supply:	Continuous flow
Resistivity:	>0.5 mega ohms
Mechanical filtration:	<0.5 µm (glycerol free) filter at discharge of deionizer
Consumption:	50 L/hour max, instantaneous demand 3.5 L/minute
Pressure:	0.49 x 10 ⁵ Pa to 3.92 x 10 ⁵ Pa (7.1–58.8 PSI)
Location:	Shutoff valve within 10 m (30 ft)
Tubing diameter:	12 mm (ID) x 18 mm (OD)
Connection:	0.5 in barbed-hose fitting

Drain Requirements:

Gravity:	Hazardous-waste floor drain Maximum height: 1.5 m (5 ft) Maximum distance from analyzer: 10 m (30 ft)
Tubing diameter:	15 mm (ID) x 22 mm (OD)

Environmental Requirements:

Average heat output:	7,200 KJ/H (6,824 BTU) max
Ambient temperature:	18–28°C (fluctuations during measurement shall be within +/-2°C)
Ambient humidity:	20%–80% relative humidity
Noise output:	Max 65 dB or less

General Characteristics:

Analytical method:	Agglutination method on terraced microplates
Channels:	12
Throughput:	300 samples/hour with 5 diluted sample cups
Sample capacity:	Capacity of 12 racks or 120 samples; continuous sample-rack loading allowed
Sample tube size:	In primary or secondary tubes: Diameter: 12–15 mm Height: 75–100 mm
Sample:	Plasma; serum; red blood cells
Reagent tray:	Up to 16 reagents can be loaded 12 positions for primary reagents (R1) 4 positions for secondary reagents (R2)
Reaction vessel:	Terraced microplates
Reaction time:	60 minutes
Assays:	ABO blood grouping, Rh typing, including weak D testing, red blood cell antigen screening, syphilis and CMV qualitative screening
Sample barcode:	NW-7; CODE39; CODE128; ISBT-CODE128; and 2 of 5 interleaved EAN-13

Dimensions:

Analyzer	(mm) 1,760 (width) x 920 (depth) x 1,380 (height) (in) 69 (width) x 36 (depth) x 54 (height)
Console	(mm) 800 (width) x 720 (depth) x 1,450 (height) (in) 32 (width) x 28 (depth) x 57 (height)
Weight:	750 kg (1,653 lbs)

CE Marked and FDA Approved

This system is class 1 laser-product compliant with IEC 60825-1:2007.

© 2022 Beckman Coulter, Inc. All rights reserved. Beckman Coulter, the stylized logo, and the Beckman Coulter product and service marks mentioned herein are trademarks or registered trademarks of Beckman Coulter, Inc. in the United States and other countries.

For Beckman Coulter's worldwide office locations and phone numbers, please visit
www.beckmancoulter.com/contact

BR-385051 | 2022-10096

